Zhermack 

Dental

universal tray adhesive

Adhesive for vinylpolysiloxane

IT - EN - FR - ES - DE - PT - NL - FI - EL - DA - NO - SV
SL - RU - PL - CS - TR - RO - HR - HU - SK - LV - LT - ET - BG

 **Zhermack S.p.A.**
Via Bovazecchino, 100 | 45021 Badia Polesine (RO) Italy
T +39 0425 597611 | F +39 0425 53596
info@zhermack.com | www.zhermack.com

Last Update -2021-10

US00082-2021-10

1. DESTINAZIONE D'USO

Adesivo per vinilpolisilossani.

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Adhesive for vinylpolysiloxanes.

2. PRODUCT DESCRIPTION

A specially formulated adhesive to ensure adhesion between addition silicones and standard, non-perforated metal impression trays and resin custom impression trays.

3. PACKAGING

10 ml bottle.

4. COMPOSITION

Polydimethylsiloxane, xylene, toluene, ethylbenzene and pigment.

5. INDICATIONS FOR USE

This device is an accessory for use with vinylpolysiloxane impression materials. It acts as an adhesive between the impression material and non-perforated metal or resin impression trays. It stops the impression material separating from the impression tray when the impression is removed from the oral cavity. This device is intended for use by dental practitioners and for the purposes of dental treatment.

6. CONTRAINDICATIONS

This device must not be used by pregnant dental personnel. In the event of accidental and unintentional contact, the presence of toluene in the material may give rise to health and safety problems in pregnant women. As a precaution, this product should NOT be used with pregnant women.

7. SIDE EFFECTS

Irritation, redness or signs of hypersensitivity may occur in the event of accidental direct or indirect contact with the device.

8. CLINICAL BENEFITS

The device stops the impression material separating from the impression tray when the impression is removed from the oral cavity.

GENERAL WARNINGS/PRECAUTIONS:

- The product's instructions for use must be kept for the duration of its use.
- These devices should only be used by dental practitioners within a professional dental environment (public or private health facilities with the necessary legal authorisations).
- Do not use the material:
 - if the lot and expiry data on the bottle's label are missing or have been deleted;
 - if variable data (lot and expiry) are no longer legible; in such case, dispose of the product as instructed in the disposal section;
 - if the primary packaging (the bottle) is damaged.
- Safety Data Sheet available subject to request.

9. STEP-BY-STEP INSTRUCTIONS

PRELIMINARY OPERATIONS:

1. Read the instructions prior to use.
2. Disinfect your hands and wear a new pair of uncontaminated disposable nitrile or vinyl gloves. Wear a face mask, protective goggles and work clothes.

PRECAUTIONS/WARNINGS REGARDING USE OF THE DEVICE:

- **DO NOT** use the product on perforated or plastic impression trays.
- The product is **NOT intended for direct contact with patients. DO NOT** use the product directly in the patient's mouth.
- **DO NOT** inhale the vapours when applying the product to the impression tray:
 - wear a face mask, protective goggles and work clothes;
 - only use the product in a well-ventilated place, well away from the operating field;
 - close the bottle immediately after use.
- **Avoid** contact with the skin and eyes. In the event of contact, rinse abundantly under running water. If irritation persists, consult a physician.
- Apply the adhesive only and exclusively to the **INSIDE** surface of the impression tray that will come into direct contact with the impression material. Take care to avoid applying it to the outside edges.
- To ensure correct use and patient safety, rigorously respect the solvent evaporation times (at least 3 or 5 minutes depending on the chosen drying method).
- If the product proves too viscous to be applied to the impression tray, dispose of it as instructed in section 11 on disposal.

- Store the product in its original, closed bottle, out of direct sunlight, in a cool, well ventilated place (5°C/41°F – 27°C/80°F), and well away from sources of heat, naked flames, sparks and other sources of ignition.
- Do not use after the expiry date indicated on the bottle label.
- Traces of dry adhesive can be removed from metal impression trays using acetone.
- **DO NOT** ingest.

USING THE DEVICE:

Handle, apply and store the product outside the operating area and well away from splashes, sprays and aerosols of body fluids:

1. Open the bottle and use the brush provided to apply a thin, uniform film of adhesive only to the inside surfaces of the uncontaminated and dry impression tray (ready for use or previously cleaned, disinfected and sterilised).
2. Close the bottle immediately after use to stop the solvent evaporating and shake it to wet the brush in the solution.
3. Dry the applied adhesive for at least 3 minutes in a gentle jet of air, or for at least 5 minutes in still air.
4. Fill the impression tray with the chosen addition silicone, taking care to cover all the areas to which the adhesive has been applied.

10. CLEANING AND DISINFECTING THE DEVICE:

This product is a device that CANNOT be cleaned and disinfected by the user. Follow these instructions carefully to minimise the risk of transmitting infectious agents:

- always wear new, uncontaminated nitrile or vinyl gloves when handling the product. If gloves become contaminated, change them immediately before handling the product;
- apply the product outside the operating field, well away from splashes, sprays and aerosols of body fluids;
- only use the product on correctly reconditioned and dry impression trays;
- close the bottle immediately after use and shake it, make sure to wet the brush in the solution;
- store the product in a closed, dry and clean place, well away from splashes, sprays and aerosols of body fluids.

In the event of any part of the product becoming contaminated during use, dispose of it as instructed in the disposal section.

11. TECHNICAL DATA

Solvent evaporation times vary with the drying method used:

- at least 3 minutes in a jet of air
- at least 5 minutes in still air

12. STORAGE, STABILITY AND DISPOSAL

The following indications for disposal are valid only if the product has been stored and used according to these instructions for use. Based on the information provided in the SDS, the waste generated by disposal of the product is hazardous to human health. It should therefore be treated as hazardous waste according to local regulations. Contaminated and non-sanitised parts that have come into contact with the patient must be disposed of as special biological waste. When handling contaminated parts, refer to chapter 10, Section Manual Cleaning and Disinfection of the device. If the device is contaminated, dispose of it as special biological waste. Always wear clean gloves when handling the device and dispose of it according to applicable local regulations.

13. IMPORTANT REMARKS

Information provided in any way, even during demonstrations, does not invalidate the instructions for use. Operators are required to check that the product is suitable for the application envisaged. The manufacturer cannot be held responsible for damage, including to third parties, deriving from failure to follow instructions or from unsuitability for an application. The manufacturer's liability is, in any case, limited to the value of the products supplied. Report any serious incident involving the medical device to the manufacturer and to the relevant authorities.

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