50 mm

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40 mm

160 mm

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OSUVIN SPRAY
Benzalkonium Chloride

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NASAL SPRAY SOLUTION

OSUVIN SPRAY
Benzalkonium Chloride

- Helps clean and sanitize the nose and throat
- Support to prevent bacteria, anti-inflammatory
- Relieves sinusitis, stuffy nose, runny nose

70 ml

INGREDIENTS:

Benzalkonium Chloride, Sodium Hyaluronate, Sodium Chloride, D-Panthenol, Menthol, Eucalyptus essential oil, Cu²⁺, Zn²⁺, Pure water.

FUNCTIONS:

- Helps clean, clean the nose and throat, remove mucus, dirt, create a feeling of ventilation, ensure natural moisture for the nasal mucosa.
- Support antibacterial, anti-inflammatory, support to reduce symptoms of the nose and throat in cases of sinusitis, stuffy nose, runny nose, pharyngitis, sore throat caused by bacterial infections or caused by allergic agents. Candlestick.

USAGE:

+ Open the product cap and press the spray nozzle out until the spray solution comes out of the nozzle to form a mist.
+ Quickly and firmly press 2-3 sprays each time into each nostril.
In case of sinusitis, nasopharyngitis due to bacterial infections, allergies: Spray 4-6 times a day.
Daily nasal hygiene: Spray 3-4 times a day.
Close the product lid.

RECOMMENDED DOSE:

Use for children 6 months old, adults, pregnant and lactating women.

STORAGE:

Keep out of reach of CHILDREN
Store in a cool, dry place, away from direct sunlight, at a temperature not exceeding 30°C.

Registration number: 220000066/PCBA-TB

Product of:
**VIET Y PHARMACEUTICAL TECHNOLOGY
JOINT STOCK COMPANY**
Address: Lot B3, Nguyen Duc Canh Industrial Park, Tran Thu Do Street, Tran Hung Dao Ward, Thai Binh City, Thai Binh Province.

Made in:
**VIET Y PHARMACEUTICAL TECHNOLOGY
JOINT STOCK COMPANY**
Address: Lot B3, Nguyen Duc Canh Industrial Park, Tran Thu Do Street, Tran Hung Dao Ward, Thai Binh City, Thai Binh Province.

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