

HƯỚNG DẪN SỬ DỤNG SẢN PHẨM LADYBALANCE



GIÁM ĐỐC
Nguyễn Đức Thọ



Intimate well-being for a more
feminine you

Please read this leaflet carefully before use.

LadyBalance[®] is a prebiotic which restores natural pH and maintains vaginal health.

Intended use: Reduces unpleasant vaginal odour and discharge associated with bacterial vaginosis (BV). Relieves vaginal dryness, itching, irritation and soreness.

Instructions for use: Insert the tablet deep into the vagina using a clean finger. Moistening the tablet may help insertion.

• **Treatment:** Use 1 vaginal tablet daily until symptoms improve (usually 1-2 weeks for low grade chronic or recurring problem, unpleasant odour and discharge).

• **Antibiotic combination treatment:** Use 1/2 -1 tablet daily after antibiotic treatment (for high grade bacterial vaginosis).

• **Prevention, maintaining natural pH and vaginal health:** Use 1/2 -1 vaginal tablet on alternate days, as required.

Contents: Lactose monohydrate, magnesium stearate and silicon dioxide.

Do not use if you are allergic to milk - contains traces of milk proteins. LadyBalance[®] can be used in cases of dietary lactose intolerance.

See your doctor if you have vaginal pain, bleeding or a yellow-green unpleasant smelling discharge - you could have Trichomoniasis.

Not recommended if you are also using vaginal products containing bacteria or yeasts (including milk-based products) - this could reduce the effect of LadyBalance[®] or cause adverse effects.

LadyBalance[®] is intended for women in all ages (not children).

LadyBalance[®] can be used during pregnancy and breast feeding but see your doctor if you think you have an infection during pregnancy. If symptoms persist, see your doctor.

LadyBalance[®] is for vaginal use only - avoid swallowing the tablet due to risk of choking.

Keep dry and out of the reach and sight of children.

Possible side effects: Clear or gritty discharge (1 in 10 users); light incidents: yeast smell, urge to pass water, headache, itching, spotting (light bleeding) (1 in 100 to 1 in 10 users). These improve after stopping treatment.

Manufacturer: LadyBalance ApS, Karensdalvej 76, 7100 Vejle, Denmark.

For more information, visit:
www.ladybalance.com

Distributor: European American Pharmaceutical Distribution Company Limited, Lot B5, No.15, 91 Lane, Nguyen Chi Thanh street, Dong Da, Hanoi.
www.eapharma.com.vn

LadyBalance[®] is a trademark of IDH Holding ApS.

