



TECHNICAL DESCRIPTION OF LADYBALANCE


| No | Heading | Content |
|-----|------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1 | Description of the medical device | |
| 1.1 | Description of the medical device | <p>This product is a vaginal tablet of 1200 (1100-1300) mg, dividable into two pieces, consisting of 96 (95,5-99,9) % lactose monohydrate, Ph. Eur grade, (0,3-2,0) % magnesium stearate, veg., Ph. Eur grade, (0,1-2,5) % silicium dioxide (Syloid AL-1 FP).</p> <p>The components are mixed and pressed into tablets without further additives or processing aids.</p> |
| 1.2 | List of accessories and fittings | 96 (95,5-99,9) % lactose monohydrate, Ph. Eur grade (0,3-2,0) % magnesium stearate, veg. , Ph. Eur grade, (0,1-2,5) % silicium dioxide (Syloid AL-1 FP). |
| 1.3 | Purpose/ Instruction | <p>LadyBalance vaginal tablets is used to treat and prevent:</p> <ul style="list-style-type: none">• Bacterial vaginosis (BV)• Unpleasant vaginal odour and discharge• Vaginal dryness and itching• Vaginal irritation and soreness |
| 1.4 | Instruction | <p>The tablet must be applied deep into the vagina by a finger. Moistening the tablet may help insertion.</p> <p>The women will use the product according to perceived need - whenever she feels an imbalance or bad smell and in periods where she by experience has problems (after the period, after unprotected sex).</p> <p>Normal dosage is ½ - 1 tablet at a time, with between 3 and 20 inserts per month. To prevent and support treatment of bacterial vaginosis, maintain a healthy vagina ½ - 1 tablet is used every day.</p> <p>LadyBalance can be used during pregnancy and breast feeding but see your doctor if you have an infection during pregnancy.</p> |
| 1.5 | Contraindications | The product should not be used by women, who are allergic to milk protein. |

| | | |
|-----|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1.6 | Warnings and cautions | <p>Do not use if you are allergic to milk - contains traces of milk proteins. LadyBalance can be used in cases of dietary lactose intolerance.</p> <p>See your doctor if you have vaginal pain, bleeding or a yellow-green unpleasant smelling discharge - you could have Trichomoniasis.</p> <p>Not recommend if you are also using vaginal products containing bacteria or yeasts (including milk-based products) - this could reduce the effect of LadyBalance or cause adverse effects.</p> |
| 1.7 | Possible bad effects | <p>Clear or gritty discharge (1 in 10 users); light incidents: yeast smell, urge to pass water, headache, itching, spotting (light bleeding) (1 in 100 to 1 in 10 users). These improve after stopping treatment.</p> |
| 2 | <p>Circulated information of product in countries (if any)</p> <p>The product is accepted by the Danish Agency for Medicine based on European Union legislation. The approval is therefore valid for the European Union.</p> | |
| 3 | <p>Registered indications in other countries (if any)</p> | |
| 4 | <p>Important safety / performance related information</p> <p>The requirement to report back any safety events is informed through the distribution channel. No significant adverse effects have been reported through over 10 years of marketing in Denmark, Europe and other countries in the world.</p> <p>This product does not contain any animal or human cells, cells, tissue or derivatives of microbiological or recombinant origin or any irradiation sources.</p> | |

Legitimate representative of product owner

Signature (Full name and title)

Confirm by stamp or digital signature



INGE DORTHE HANSEN
CEO; DEVELOPER






This is to certify that **Inge Dorte Hansen** today in my presence at the Notarial Office approved and signed the above document. No conspicuous corrections or addenda were found in the document.

Inge Dorte Hansen has proved her identity by showing passport.

The Court in Kolding denmark, 2022-03-08

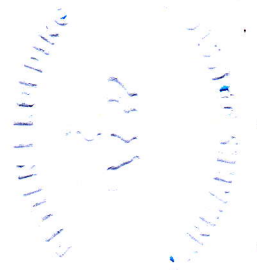Charlotte Rytger
Charlotte Rytger
Notary Public




| APOSTILLE (Convention de La Haye du 5 octobre 1961) | | | |
|---------------------------------------------------------------|------------------------------------------------------------------------------------|------------------------------------------|----------------------------------------------------------------------------------------------------------|
| 1. Country: Land: | | Denmark Danmark | |
| This public document Dette offentlige dokument | | | |
| 2. has been signed by er underskrevet af | | Charlotte Rytger | |
| 3. acting in the capacity of i egenskab af | | Notary Public Notar | |
| 4. bears the seal/stamp of er forsynet med segl/stempel af | | The Court in Kolding Retten i Kolding | |
| Certified Attesteret | | | |
| 5. at i | Copenhagen København | 6. the den | 09 Mar 2022 09 mar 2022 |
| 7. by af | Ministry of Foreign Affairs of Denmark Udenrigsministeriet | | |
| 8. No nr. | 6831AE63 | | |
| 9. Seal/stamp: Segl/stempel: |  | 10. Signature: Underskrift: |  Lene Møller Hansen |

This Apostille only certifies the authenticity of the signature and the capacity of the person who has signed the public document, and, where appropriate, the identity of the seal or stamp which the public document bears. This Apostille does not certify the content of the document for which it was issued.

To verify the issuance of this Apostille, scan the QR code or visit the following website:

<https://e-register.um.dk>

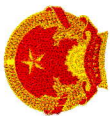

ĐẠI SỨ QUÁN NƯỚC CHXHCN VIỆT NAM TẠI ĐAN MẠCH
EMBASSY OF THE S.R. OF VIET NAM IN DENMARK

CHỨNG NHẬN/HỢP PHÁP HÓA LÃNH SỰ
CONSULAR AUTHENTICATION

- VIỆT NAM
Viet Nam
Giấy tờ, tài liệu này
This public document
2. do Ông (Bà) **LENE MØLLER HANSEN** ký
has been signed by **LENE MØLLER HANSEN**
3. với chức danh **Cán bộ hành chính**
acting in the capacity of **Administrative Officer**
4. và con dấu của **BỘ NGOẠI GIAO ĐAN MẠCH**
bears the seal/stamp of the **Ministry of Foreign Affairs of Denmark**
được chứng nhận/hợp pháp hóa lãnh sự
Certified
5. tại **COPENHAGEN**
at **Copenhagen**
6. ngày **9/3/2022**
9 March 2022
7. Cơ quan cấp: **ĐẠI SỨ QUÁN NƯỚC CHXHCN VIỆT NAM**
TẠI ĐAN MẠCH
by **Embassy of the S.R. of Viet Nam in Denmark**
8. Số: **30.03/HPHLS/22**
No: **30.03/HPHLS/22**

Bí thư thứ nhất / First Secretary

