

INSTRUCTION MANUAL

MABC0016

**BASSINET CART
MODEL : PB-1100**

PREFACE

Thank you for purchasing Paramount Bed's Bassinet Cart (PB-1100)

This Instruction Manual describes precautions and how to safely use and operate the Bassinet Cart.

- Before using this Bassinet Cart, please read Instruction Manual to fully understand the safe and proper operating procedures.
- The actual product purchased may differ from detailed descriptions given in this Instruction Manual due to product improvements.
- For any queries, please feel free to directly contact your dealer or Paramount Bed.

PURPOSE OF THE BED

This bed is designed for being used for newborn baby under the supervision by facility's staff such as doctors and nurses.


CAUTIONS TO TAKE IN USE

- Keep this Instruction Manual handy for reference.
- Do not change or modify anything of the bed.

HOW TO USE


1. Installation of Parts

- Please install the parts as shown in the drawing

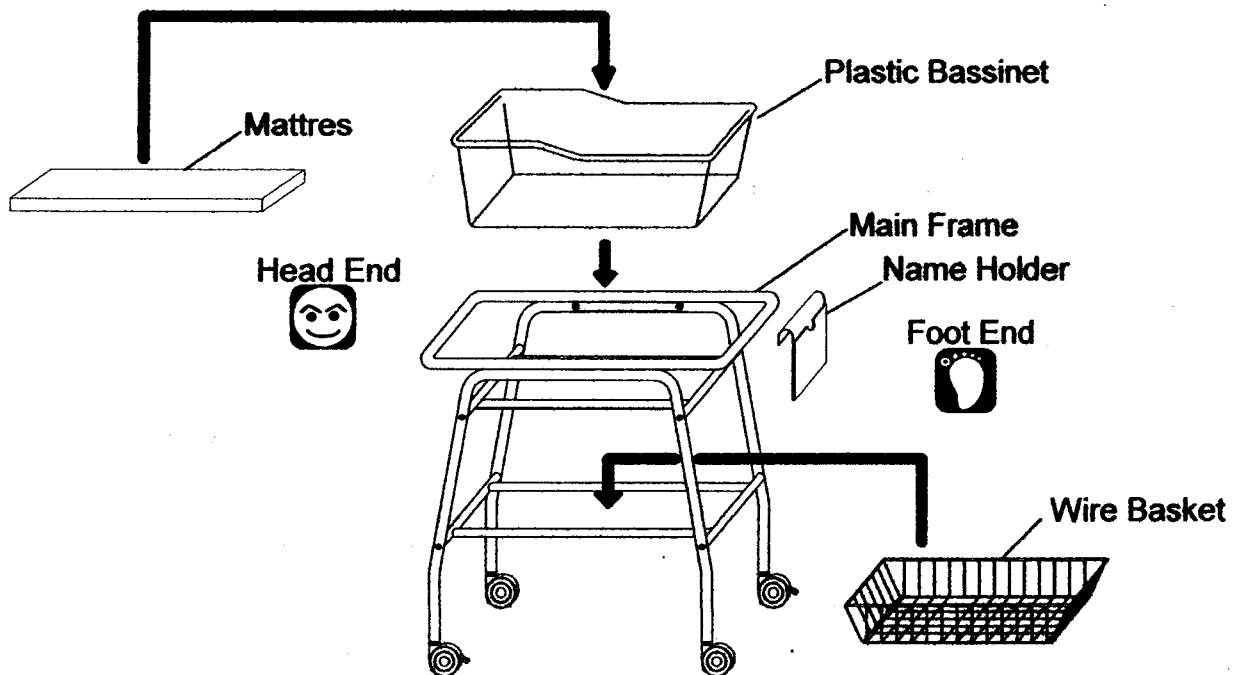

2. Caster

- Please use the stopper as shown in the right drawing and lock caster firmly

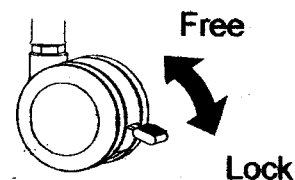

SAFETY PRECAUTIONS

⚠ WARNING


- Indicates precautions on action that may result in fatal injury if ignored and misused

- Do not transfer the Cart on uneven surface
- Lock the casters in daily use
- When transferring bed, do not move on a path or hallway with substantially different levels or on extremely uneven surface.
- If you move the bed under bad conditions, move it as slowly as possible.
- Always apply the brake of the casters except moving the bed. Otherwise, the cart easily move and it is dangerous.

⚠ WARNING

- Please keep two casters locked with stoppers when you do not transfer the bed.

DAILY MAINTENANCE AND CLEANING

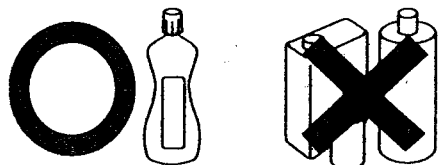

- Never use solvent. Such as cresol, thinner, benzene or gasoline for cleaning.
- For cleaning, please use a cloth soaked in diluted detergent and wrung out well and then wipe off with dry cloth.

SPECIFICATION

DESCRIPTION		BASSINET CART
MODEL		PB - 1100
DIMENSIONS (mm)		755 (L) x 480 (W) x 868 (H) mm
WEIGHT (kg)		14 Kg
MATERIALS	MAIN FRAME	Steel pipe, powder coating finishing
	WIRE BASKET	Steel wire, powder coating finishing
	PLASTIC BASSINET	Clear polycarbonate
	MATTRES	Polyurethane foam
CASTERS		Ø75mm (Free x 2pcs, With stopper x 2pcs)

Produce by : PT. PARAMOUNT BED INDONESIA
Address : MM2100 INDUSTRIAL TOWN, BLOCK M-1-1
EXPORT PROCESSING ZONE, CIKARANG BARAT
BEKASI 17520, INDONESIA
TEL: 62-21-8981389 FAX: 62-21-8980716