

**NHÃN – BAO BÌ ĐÓNG GÓI**  
**Tên sản phẩm: DÂY THỞ**

**VADI Neonatal Breathing Tubing**

**REF G-329000**

**Contents: customized silicone tubing (30-180cm in length);  
water trap; connector; Quantity by demands.**

**Usage :**

1. Sterilization before usage.
2. Connect 2 customized silicone tubes onto one water trap.
3. Then connect customized silicone tubes with water trap to wye connector.
4. Connect tube to the inlet of humidification chamber and the other to the ventilator.
5. Connect the one limb of breathing circuit to the outlet of chamber.
6. Connect the rest tube to the expiratory port of ventilator.
7. Perform the leak test of whole set of breathing circuit according to the individual operating manual of ventilator.


**Sterilization:**

1. Dismantle all components except the stainless steel spring attached to the body of water trap
2. Clean the parts with low flow running water, using a soft brush to remove all foreign material. (for example, soil or organic matter)
3. Sterilization by autoclave: 121°C/15PSI/30minutes. (by 2008 CDC Guideline)
4. Allow autoclave maximum 20 times.

**Caution**

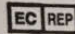
1. Check all connects are tight before use.
2. DO NOT take the spring-loaded valve off the water trap Connector.
3. This product is used under physician's order.
4. Please dispose it according to the local rules of the disposition of medical device/accessory waste.



 **VADI Medical Technology Co., Ltd.**

Address : 5F-2, No.492-19, Sec. 1, Wan Shou Rd., Kwei Shan Hsiang, Taoyuan, Taiwan.

TEL : +886-2-8209-2833 E-mail : info@vadimedical.com.tw

 **Obelis S.A.**

Address : Bd. Général Wahis 53, 1030 Brussels, Belgium.

TEL : +32-2-732-5954 FAX : +32-2-732-6003 E-mail : mail@obelis.net

**Expired Date : 3 years after manufacture date.**



Please see the seal.



Please see the seal.

**CE 0434**

**Made In Taiwan**  
X-3028-03.EN(Ver.2014-01A)