

NoseFrida[®]

Nasal Spray

Saline solution
for children
aged 0-3



**Child-friendly dosage
Extra long shelf life
Natural sea salt solution**

NoseFrida®

NoseFrida® Nasal Spray is a saline solution made from natural sea salt. One dose gives 0.07 ml of solution, and has been specially developed for babies and young children. The size of the spray bottle top has also been designed especially for small children. The nasal spray is effective for blocked noses, and is suitable for use together with the **NoseFrida® Nasal Aspirator**. It can be used daily, does not contain any decongestants, and is free from preservatives.

Thanks to its unique pump function, it has an extra long shelf life after opening.

Once opened, the pack should only be used for one child.

Keep out of the reach of children.

CE



7 330304 424251

Volume: 20 ml

NoseFrida®

Ingredients: Water, 0.9% natural sea salt (isotonic solution), PH neutral.

Shelf life: Unopened packaging 5 years, opened packaging 4 months.

Dosage: 1-2 doses per nostril.

Instructions for use: Hold the child upright, pump once or twice into the air to ensure an even spray, and spray once or twice into each nostril. If necessary, use the **NoseFrida® Nasal Aspirator** (sold separately) to clear the nostrils.



Waste sorting: Outer box as paper packaging, entire spray container and lid as rigid plastic.

Manufacturer: Nasalprödnukter Sverige AB,
S:t Johannisgatan 2, 211 46 Malmö,
Sweden. Tel: +46 40 699 25 20

Volume: 20 ml