**COMPOSITION:**  
Propolis, Papain, Peppermint oil, Anise oil, Fennel oil, Sodium chloride, Sodium saccharin, Sucralose, Cremophor RH40, Sodium benzoat, Purified water sqf 15ml.

**USES:**  
- Supports relieve cough, itchy and buring throat.  
- Supports reducing bacteria, ameliorate symptoms of inflammatory affections of the buccal and pharyngeal cavity, dental caries and bad breath.

**USE FOR:**  
- People with a cough, buring throat, sore throat.  
- People with symptoms of inflammatory affections of the buccal and pharyngeal cavity, the halitosis.

**DIRECTIONS:**  
**Use for baby over 6 months:** Apply the sprayer 2-4 times daily to the affected region of the buccal and pharyngeal cavity, 2-4 puffs per times. Can increase times use per day if necessary.  
**Sore throat, buring throat, cough:** Spray deep into the throat.  
**To prevent dental caries and halitosis:** Spray into the oral cavity and the tooth.

**Caution:**  
- Not use for people who be allergic to any components of the product.  
- Only use within 30 days after the first opening the cap.

**Storage:** Store in a cool dry place, protected from light, temperature under 30 degrees C  
**Package:** Box of 1 bottle 15ml of spray solution.

Manufacturer:  
**DƯỢC KHOA PHARMACEUTICAL JOINT STOCK COMPANY**  
Head office: No 9 Nguyễn Công Trứ St., Phạm Đình Hổ, Hai Bà Trưng Dist., Hanoi, Vietnam.  
Manufacturing address: DKPharma Pharmaceutical Factory, Bac Ninh branch of Duoc Khoa Joint Stock Company, Lot No.III-13, D3 Street, Que Vo II Industrial park, Ngọc Xá commune, Que Vo District, Bac Ninh Province.

PropoBee Baby SPRAY

PropoBee Baby SPRAY

Reduce symptoms of cough sore throat

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**COMPOSITION:**  
Propolis, Papain, Peppermint oil, Anise oil, Fennel oil, Sodium chloride, Sodium saccharin, Sucralose, Cremophor RH40, Sodium benzoat, Purified water sqf 15ml.

**Uses, use for, directions, ... see the enclosed leaflet.**

**CAUTION:** Not use for people who be allergic to any components of the product.  
Only use within 30 days after the first opening the cap.  
**STORAGE:** Store in a cool dry place, protected from light, temperature 30 degrees C.

0.5 fl or (15ml) Sterile

Manufacturer:  
**DƯỢC KHOA PHARMACEUTICAL JOINT STOCK COMPANY**  
Head office: No 9 Nguyễn Công Trứ, Phạm Đình Hổ, Hai Bà Trưng Dist., Hanoi, Vietnam.  
Manufacturing address: DKPharma Pharmaceutical Factory, Bac Ninh branch of Duoc Khoa Joint Stock Company, Lot No.III-13, D3 Street, Que Vo II Industrial park, Ngọc Xá commune, Que Vo District, Bac Ninh Province.

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## PropoBee Baby SPRAY

### COMPOSITION:

Propolis, Papain, Peppermint oil, Anise oil, Fennel oil, Sodium chloride, Sodium saccharin, Sucralose, Cremophor RH40, Sodium benzoat, Purified water sqf 15ml.

### USES:

- Supports relieve cough, itchy and buring throat.
- Supports reducing bacteria, ameliorate symptoms of inflammatory affections of the buccal and pharyngeal cavity, dental caries and bad breath.

### USE FOR:

- People with a cough, buring throat, sore throat.
- People with symptoms of inflammatory affections of the buccal and pharyngeal cavity, the halitosis.

### DIRECTIONS:

**Use for baby over 6 months:** Apply the sprayer 2-4 times daily to the affected region of the buccal and pharyngeal cavity, 2-4 puffs per times. Can increase times use per day if necessary.

**Sore throat, buring throat, cough:** Spray deep into the throat.

**To prevent dental caries and halitosis:** Spray into the oral cavity and the tooth.

### CAUTION:

- Not use for people who be allergic to any components of the product.
  - Only use within 30 days after the first opening the cap.
- STORAGE:** Store in a cool dry place, protected from light, temperature under 30 degrees C

**PACKAGE:** Box of 1 bottle 15 ml of spray solution

Manufacturer:  
**DUOC KHOA PHARMACEUTICAL JOINT STOCK COMPANY**  
Head office: No 9 Nguyen Cong Tru St., Pham Dinh Ho, Hai Ba Trung Dist., Hanoi, Vietnam.  
Manufacturing address: DKPharma Pharmaceutical Factory  
Bac Ninh branch of Duoc Khoa Joint Stock Company:  
Lot No.III-1.3, D3 Street, Que Vo II Industrial park,  
Ngoc Xa commune, Que Vo District, Bac Ninh Province.